



Dermaroller[®]



Genuine Dermaroller[™] Therapy[™]
Natural Skin Regeneration and Scar Repair
INFORMATION



Natural Skin Regeneration and Scar Repair Therapy

Genuine Dermaroller™ Therapy can significantly improve the appearance of:

- Acne scars
- Aged and sun damaged skin
- Facial and décolleté lines and wrinkles
- Stretch marks

Genuine Dermaroller™ Therapy provides advanced micro-medical skin-needling, a procedure that stimulates the skin to regenerate and repair itself naturally and safely, creating smoother, brighter, healthier, younger-looking skin.



Genuine Dermaroller™ Therapy™ utilises sterile single-use medical devices.

Quality and safety are guaranteed.

Genuine Dermaroller™ Therapy is only available from trained medical practitioners and authorised clinics.

Do not accept substitutes, only have Genuine Dermaroller™ Therapy carried out by a trained medical practitioner at an authorised clinic using Genuine Dermaroller™ medical devices.

Details of trained practitioners and authorised clinics are available from AesthetiCare® on: **0800 0195 322** or www.genuinedermaroller.co.uk and you can find us on: www.facebook.com/genuinedermaroller and you can follow us on twitter: [@dermarolleruk](https://twitter.com/dermarolleruk)

Genuine Dermaroller™ Therapy™ Procedure


Your practitioner will use a Genuine Dermaroller™ medical device to produce thousands of micro-medical needle-columns in the skin. Each column will penetrate into the dermis of the skin and will be approximately 0.1mm in width. The column will close rapidly enabling the skin to recover quickly - often in the same day. Your practitioner will use a topical anaesthetic to numb the skin making the procedure comfortable; it will normally take up to 30 minutes for the topical anaesthetic to work and approximately 20 minutes for the Genuine Dermaroller™ Therapy procedure.

After a Genuine Dermaroller™ Therapy procedure your body will start to naturally regenerate and repair the skin, working below the surface in the dermis. This takes time as new collagen is formed, new skin cells are generated and blood supply is enhanced. It can take up to 6 weeks before visible signs of regeneration and repair are seen and the process will continue over the following months, providing you with a natural and long lasting enhancement.


Your practitioner will normally recommend that your Genuine Dermaroller™ Therapy consists of a series of 3 Genuine Dermaroller™ procedures with approximately 6 weeks between them.

In the treatment of acne scars and other types of scarring it may be necessary for your practitioner to extend your Genuine Dermaroller™ Therapy.

Skin Biopsy images showing new collagen

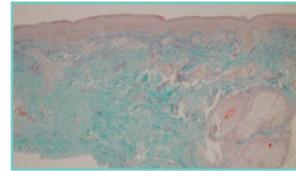

Before

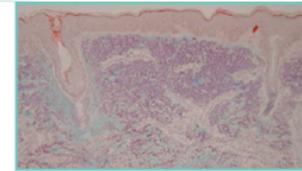

After

Before and after Genuine Dermaroller™ Therapy. The area stained purple in the skin biopsy taken after a Genuine Dermaroller™ procedure is newly formed collagen.



Acne scarring

Improvement after two Genuine Dermaroller™ procedures



Ageing & sun damaged skin/lines & wrinkles

Improvement 3 months after one Genuine Dermaroller™ procedure



Stretch marks

Improvement after four Genuine Dermaroller™ procedures



Efficacy and safety in pigmented skin

Improvement of acne scars after four Genuine Dermaroller™ procedures

Genuine Dermaroller™ Therapy™

- after procedure skincare



Skin immediately after Genuine Dermaroller™ Therapy procedure.

Immediately after your Genuine Dermaroller™ Therapy you will look as though you have moderate sunburn and your skin may feel warm and tighter than usual. This will start to subside after 1 to 2 hours and will normally recover within the same day with only slight redness the following day.

Your practitioner may well apply Chiroxy Cream after the procedure to help soothe and calm the skin, continue to use until the redness has cleared.

Use tepid water to cleanse the face until the redness has gone, and dry gently, always make sure your hands are clean when touching the treated area. It is recommended that make up is not applied for 12 hours after the procedure although your practitioner may be able to supply you with specialist mineral make-up that they feel would be suitable for use during this period.

If you will be in daylight immediately after the procedure apply a broad spectrum UVA/UVB sunscreen with a SPF50.





AesthetiCare® recommend Heliocare Cream or Gel, continue to apply this on a daily basis for at least two weeks even during the winter or on cloudy days.

In the days following your Genuine Dermaroller™ Therapy as the skin starts to regenerate and repair it may well feel drier than normal.

Genuine Dermaroller™ home-care epidermal skin regeneration


Once the redness has cleared after a Genuine Dermaroller™ Therapy procedure, it is important that the skin is cared for on an ongoing daily basis with an effective anti-ageing regime. An effective regime will typically use dermo-cosmetic products to

- restore the essential skin vitamins and anti-oxidants that are lost as we age

- moisturise and rejuvenate the skin
- provide protection from UV light

Ask your medical aesthetic practitioner what products they would recommend for your skin.

Genuine Dermaroller™ home-care epidermal skin regeneration uses the HC9 Dermaroller™ device. Used 2 to 3 times a week at home under the direction of your aesthetic practitioner, it will stimulate the epidermis and enhance the penetration of specific skin products, improving your anti-ageing skincare and your goal of younger and healthier looking skin.

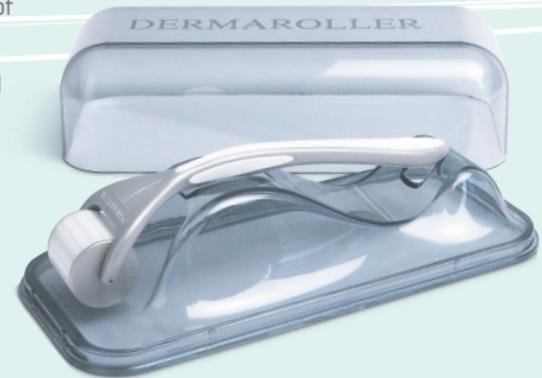

The Genuine Dermaroller™ Therapy programme is only available from trained medical practitioners at authorised Dermaroller™ clinics. Authorised Dermaroller™ clinics only use Genuine Dermaroller™ sterile single-use medical devices.

Dermaroller™ is a trademark of Dermaroller GmbH.

Photographs show the results after procedures with Genuine Dermaroller™. Individual results can differ.

www.genuinedermaroller.co.uk

Freephone 0800 0195 322



Follow us on twitter @dermarolleruk



Find us on Facebook at:
www.facebook.com/genuinedermaroller

Advanced dermo-cosmetic and dermatological research-based products

www.aestheticare.co.uk

Unit 124, Thorp Arch Estate, Wetherby,
West Yorkshire LS23 7BJ

A division of Ferndale Pharmaceuticals Ltd.

06.11. GD Patient Leaflet v1

Your Authorised Genuine Dermaroller™ Clinic




The Professional Choice